ME Cupping Set: Complete Care & Troubleshooting Guide (EN)

Revision date: April 13, 2026

Section 1: Cleaning & Disinfection

Maintain hygiene standards based on your usage level.

Daily Cleaning (Home Use)

1. **Disassemble:** Pull the **blue silicone valves** out from the top of the cups.
2. **Wash:** Use warm water and mild liquid soap. Ensure all massage oils or skin creams are removed.
3. **Sanitize:** Wipe the cups thoroughly with a cotton ball soaked in **70% Isopropyl Alcohol (Methylated Spirits)**.
4. **Dry:** Air-dry completely before reassembling.

High-Level Disinfection (Clinical/Professional Use)

If the cups are used in a professional setting or have come into contact with body fluids:

- **Hydrogen Peroxide (H₂O₂):** Soak in a **6% to 7.5% solution** for **30 minutes**.
 - *Note: If using 12% Food Grade H₂O₂, mix 1 part peroxide with 1 part distilled water.*
- **Bleach Solution:** Use a **1:10 dilution** of household bleach (as per US CDC guidelines) for 10 minutes.
- **Autoclave:** Safe for the polycarbonate cups only. **Remove blue silicone valves first** to prevent premature deterioration.

Section 2: Troubleshooting (Common Issues)

Issue	Possible Cause	Solution
Weak Suction	Silicone valve is stuck or dry.	Pull the blue valve up slightly to "prime" it, or apply a tiny drop of vegetable oil to the valve stem.
Cup Won't Stay On	Skin is too dry or hairy.	Apply a small amount of massage oil or lotion to the skin to create a better airtight seal.
Pump Not Pulling	Pump nozzle is loose.	Ensure the pistol pump is pushed firmly and straight onto the blue valve before pulling the trigger.
Cracked Cups	Harsh chemicals used.	Please replace any cracked cups immediately. Avoid "Technical Grade" cleaners or industrial solvents which can make the polycarbonate brittle.

Section 3: Clinical Safety Checklist

- **Valve Maintenance:** Check the silicone blue valves regularly. If they become hard or cracked, they should be replaced to maintain suction.
- **Skin Integrity:** Never use cupping on:
 - Open wounds, inflamed skin, or fresh scars.
 - Patients with pacemakers or active internal bleeding.
 - Pregnant women (consult a doctor first).
- **The "Cupping Mark":** Circular red or purple marks are a normal and expected occurrence following treatment. These marks typically fade away within **3–10 days**.

Section 4: Storage

Store your ME Cupping Set in a cool, dry place. Avoid leaving the cups in direct sunlight for extended periods, as UV rays can damage the high-quality polycarbonate and silicone components.

For further assistance or to inquire about replacement components (cups and valves), please contact our Customer Support team.

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